General Guidelines For Use

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Sani-
GERMICIDAL WIPE
Toallitas Germicidas

Use on hard, nonporous environmental surfaces.
Uselas en superficies ambientales duras y no porosas.

1. Remove lid and discard inner seal. Find center of wipe roll, remove first wipe for use, thread next wipe through the hole in the canister lid. Pull through at least one inch. Replace lid. To secure wipe, pull wipe down into small opening.

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- 2a. Cover the opening half way with one hand. Remove wipe(s) with a uniform pull away from face and eyes.
2b. Dispense single wipes as necessary by pulling out at an angle through the small opening.

- 2c. Dispense multiple wipes as necessary by pulling vertically through the large opening. Pull down into small opening to tear.

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- 3a. If present, use a wipe to remove visible soil prior to disinfecting.

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- 3b. Unfold a clean wipe and thoroughly wet surface.

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4. Allow surface to remain visibly wet for one (1) minute. Use additional wipe(s) if needed to ensure continuous 1 minute wet contact time. Let air dry.

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5. Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet.

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PDI, Be the Difference, Sani-HP1, and all associated logos and product images are trademarks or properties of Professional Disposables International, Inc.

©2025 PDI PDI04259142
400 Chestnut Ridge Road, Woodcliff Lake, New Jersey 07677
800.999.6423 pdihc.com